Rear Wheel Drive


Featuring:


- · Amigo Centra
- Amigo Deluxe
- Amigo Sport
- Amigo IV



Welcome and First Things First	2, 3
Safety First	
■ Safety Suggestions	
Getting Down to Basics	
Assembly Instructions	
Now It's Your Turn	
■ Operating Your Amigo	
■ Helpful Hints for Everyday Use	
Amigo, A Friend for Life	
■ Maintenance	
■ Service	
■ Warranties	25 - 27
Accessories and Service Kits	28, 29
Amigo Centra (Exploded View Diagrams)	
(Parts List)	
Amigo Deluxe (Exploded View Diagrams)	34, 35
(Parts List)	36, 37
Amigo Sport (Exploded View Diagrams)	
(Parts List)	
Amigo IV (Exploded View Diagrams)	
(Parts List & Wiring Diagrams)	
Wiring Diagrams (Amigo Centra)	
(Amigo Deluxe)	
(Amigo Sport)	
Amigo Step-Up Program	45

CONTENTS WWW.

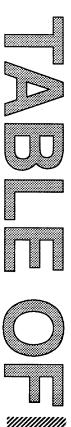

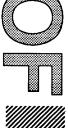





Allan R. Thieme

elcome to the Amigo family of active and independent people. For thousands of individuals around the world, the AMIGO provides the opportunity to enjoy life more fully. We are confident that the following instructions and helpful hints will make the Amigo an extremely useful and reliable "friend." Because we are concerned about your health and safety, we ask that you read this manual carefully before operating your Amigo. When you learn how to use the Amigo properly, you will discover a world of enjoyment and independence.

As a member of the Amigo family, you will receive the *Friendly Wheels*, our magazine written especially for and about Amigo owners. The *Friendly Wheels* will keep you informed about new products and accessories, helpful hints, service tips, special offers and much more!

Please write us and include a photograph of you and your Amigo. Your letters are a source of inspiration to us. As you use your Amigo, be sure to share your helpful hints and operating suggestions with us so we can pass them on to other owners.



Wishing you happiness and good health from all of us at Amigo Company.

Sincerely,

FIRST
President & Founder

THINGS FIRST

- **CHECK YOUR ORDER**
- COMPLETE AND RETURN WARRANTY CARD
- DO NOT OPERATE YOUR AMIGO UNTIL YOU HAVE COMPLETELY READ THIS OWNER'S MANUAL
- **REPAIR HOTLINE 1-800-248-9131**

For your convenience and understanding, signal words are defined below:

△ DANGER

Danger is used to indicate the presence of a hazard which will cause severe personal injury, death, or a substantial property damage if the warning is ignored.

AWARNING

Warning is used to indicate the presence of a hazard which can cause severe personal injury, death, or substantial property damage if the warning is ignored.

▲ CAUTION

Caution is used to indicate the presence of a hazard which will or can cause minor personal injury or property damage if the warning is ignored.





■ ASSEMBLY INSTRUCTIONS

- Handle
- Seat installation
- Seat Positioning
- Dual Wheels

■ BATTERY INFORMATION AND CHARGER INSTRUCTIONS

- Battery Maintenance
- · General Maintenance
- · Gel Cell Batteries
- Quick Battery Recharging Instructions
- XTENDER™ Battery Charger Troubleshooting
- CIRCUIT BREAKER
 AND FUSE REPLACEMENT INSTRUCTIONS

DOWN TO BASICS



Assembly Instructions

• HANDLE ADJUSTMENT

ALL RWDs:

To adjust the steering handle position, place the steering handle in the desired location. Tighten the button head or allen head screw located on the front of the handle with the Allen wrench provided (Figure 1).

To adjust the angle of the handle, loosen the lower knob to separate the meshed teeth. Move the handle into the desired position and tighten the knob (Figure 2).

SEAT INSTALLATION

ALL RWDs:

- 1. Unlock the seat lock.
- 2. Insert the upper seat post into the

- opening of the lower seat post.
- 3. Spin seat around to make sure it rotates freely.
- 4. To lock the seat, pull the seat lock lever toward the rear of the seat.

Note: The following information pertains to Amigos that do not have a Power Seat Lift.

To raise or lower your seat height, remove the nut and bolt that holds the seat collar in place on the seat post, then adjust and reinstall in one of the holes on the seat post.

▲ CAUTION

It is recommended that the seat lock be installed in the third hole positioned from the seat. If you desire to put your seat higher than this, you must have dual wheels on your Amigo.

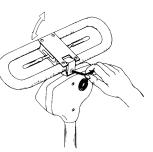


Figure 1



Figure 2

convenience. Looking at the light when the unit is not moving or when it has just started moving will not give an accurate reading.

AMIGO CENTRA:

A red indicator light, located on the top of the handle, near the switch, will let you know:

WHEN THE POWER SWITCH IS "ON"-The red indicator light will glow.

AMIGO DELUXE, AMIGO SPORT & AMIGO IV:

A battery gauge meter is located on your handle. When the key switch is turned to the ON position, the needle should be in the green area. If the needle is in the red area, you should recharge.

Charging Procedures for All Units:

Connect the cord to the socket in the rear cover of your Amigo and then to the wall outlet. To see the charge lights, look through the window on top of the rear cover. The red light should be steady, indicating the batteries are charging. The green light will glow steadily when the batteries are 90% charged. When the green light starts blinking, this indicates that the batteries are fully charged.

△ CAUTION

Make sure that the wall cord is unplugged from the wall outlet before disconnecting or moving your Amigo.

XTENDER™ Battery Charger Troubleshooting:

Problem #1. XTENDER™ does not turn on (no red light) when the extension cord is plugged into the wall outlet.

Check if the extension cord is plugged in properly to a "live" wall outlet and that the extension cord is not damaged. If cord is damaged, take unit to an authorized service dealer for repair. Do not use the XTENDER™ with a damaged cord. If power is going to the XTENDER™ and the red light does not come on, contact the Amigo Service Department.

Problem #2. XTENDER™ does not charge batteries:

If red and green LED turns on immediately:

a. Check battery connection for



loose or open connections.

- b. Check circuit breaker.
- c. If a & b check out all right, return unit to factory or contact the Amigo Service Department.

If green LED does not come on within 12 hours and unit is warm:

 a. Something is wrong with the batteries, have them checked by an Amigo factory-authorized service technician.

Problem #3. If green LED comes on but does not flash after being on six hours:

a. Return XTENDER™ to authorized dealer for service.

Problem #4. If green LED does not turn on immediately after XTENDER™ is plugged into the wall outlet and output cable is not connected to anything:

a. Return XTENDER™ charger to your

local Amigo representative or the Amigo Service Department.

Problem #5. To check for popped circuit breaker.

▲ CAUTION

To avoid battery damage, remove the batteries!

Remove your batteries. Remove the longer charger wire from the battery box by removing the screw that holds it. Plug the charger into the wall outlet - both red and green lights should be lit. Touch the terminal of the longer charger wire to the terminal of the shorter charger wire (still attached to the battery base).

- a. If this connection makes a spark and the lights on the charger dim or go out, the charger is operating properly.
- b. If this connection produces no spark and the charger lights stay lit, your fuse most likely needs to be replaced (Contact the Amigo Service Department to order a fuse).



• CIRCUIT BREAKER & FUSE REPLACEMENT **INSTRUCTIONS**

AMIGO CENTRA, **AMIGO DELUXE and AMIGO IV:**

The circuit breaker is under the rear housing and attached to the positive terminal in front of the battery holder.

AMIGO SPORT:

terminal of the battery.

ALL RWDS:

There is a circuit breaker located on top of


the onboard charger (Fig. 5). If the charger

does not operate, follow the instructions in the "Troubleshooting Guide" enclosed in your

owner's packet and reset the breaker.

A circuit breaker protects the wiring and battery in the event of a short circuit. If your Amigo stops, check for overloads or short circuits before continuing to operate.

The circuit breaker is located under the battery cover and attached to the positive

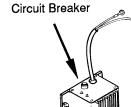

XTENDER™

Figure 5

■ OPERATING YOUR **AMIGO**


- Handle Adjustments
- Driving Your Amigo
- Manual Pushing
- Power Seat Lift **Operation - Optional**
- Power Seat Lift Maintenance
- Optional Equipment
- Transporting Your Amigo
- Airplane Travel
- Ground Travel

■ HELPFUL HINTS FOR **EVERY DAY USE**

- Bathroom
- Bedroom
- Kitchen
- · Car
- Inclines and Ramps
- Using An Elevator
- Opening Doors
- · Keep Your **Amigo Shining**
- Speed Controls
- Stopping





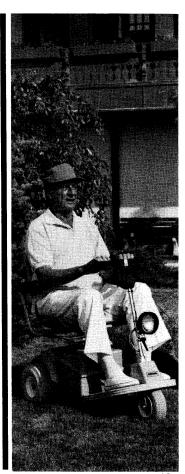

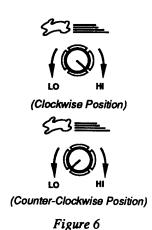






Figure 7



Operating Your Amigo

• HANDLE CONTROLS

ALL RWDs:

HANDLE CONTROLS: See "Handle Adjustment" section (page 7) for your handle.

TELESCOPIC HANDLE: The Amigo Telescopic Handle can be adjusted from 19" to 23". It is factory set to 23 inches. If adjustment is needed, please see your local Amigo Dealer or call Amigo Mobility International, Inc.

AMIGO CENTRA:

Controls include speed, power and battery indicator light. The jack will turn the Amigo ON and the speed may be adjusted with the potentiometer knob. By turning this knob to the HI position, the maximum speed obtainable is 5.3 m.p.h. The LO position of

this knob limits the maximum speed to a creep.

AMIGO DELUXE, AMIGO SPORT & AMIGO IV:

Controls include speed, battery indicator gauge, horn, key, and power seat lift. The key will turn it ON and the gauge will indicate the capacity of your batteries. Turning it OFF will prevent drainage of batteries and unauthorized use. Speed adjustment can be regulated by the control located on the handle (Figure 6). By turning it clockwise, the speed increases from 0 to 4.5 m.p.h. on the Amigo Deluxe, or 5.3 m.p.h. on the Amigo Sport and Amigo IV and when turned counterclockwise, decreases the speed from 4.5 on the Amigo Deluxe, or 5.3 on the Amigo Sport and Amigo IV to 0 m.p.h. on both models.

• DRIVING YOUR AMIGO

ALL RWDs:

To move forward, pull the right side of the hand control lever toward you (Figure 7). You can also move forward by pushing the left side of the control lever away from you. By releasing the control lever, you will come to a stop approximately one second after release. To operate in reverse, pull the left side of the hand control lever toward you, or push the right side control lever away from you. Because the AMIGO DELUXE comes equipped with electromagnetic braking, there is no manual, hand-operated brake lock.

• MANUAL PUSHING

ALL RWDs:

For safety and security, your unit is equipped with an electromagnetic brake which makes pushing your unit almost impossible. When it is necessary to push your unit, push the brake release switch to "Freewheeling" (Figure 8).

Note: When you are through pushing your unit, push the switch back to "Normal." Leaving the

switch in the "Freewheeling" position will drain the power of the battery. Also, when your unit is not in use, make sure the switch is in the "Normal" position. This will extend the life of your batteries.

∆WARNING

Do not leave switch in "Freewheeling" mode.

• POWER SEAT LIFT OPERATION -

(Optional on Amigo Centra, Amigo Sport and Amigo IV))

The Power Seat Lift controls the seat from 17 to 23 inches from the platform. In the highest or lowest position, a Slip Clutch is activated causing a clicking noise which is normal (Seat must be in lowest position when unit is in motion).

AMIGO CENTRA:

1. Move the lift switch located under the seat up or down to raise or lower it.

AMIGO DELUXE, AMIGO SPORT & AMIGO IV:

1. Push the PSL Switch (located on the handle) up to raise your


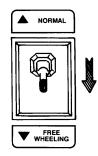


Figure 8



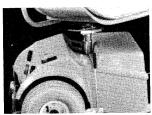

- seat to the desired height.
- 2. Push the PSL Switch down to lower your seat to the desired height.

• POWER SEAT LIFT MAINTENANCE

ALL RWDs:

Approximately every 1 - 2 months, remove the seat and activate seat lift to the highest position. Using a dry cloth, wipe inner tube of accumulated grease and dirt. Then, apply a very thin coating of grease to inner tube, except for one inch on top of tube.

Note: A tube of grease has been supplied. White Lithium grease is recommended and can be purchased at your local hardware store.

• OPTIONAL EQUIPMENT

AMIGO DELUXE, AMIGO SPORT & AMIGO IV:

The lighting package includes a front operational headlight mounted on the handle, rear operational lights (red) and the rear directional lights (yellow). The opera-

tional lighting system activated by pushing the "light" switch to the ON position. This switch is found on the left side of the handle control box, next to the meter, and is clearly marked "lights."

The rear directional lights can be activated by moving the "directional" switch on the control box. This switch, marked "directional," is found on the right side of the control box, next to the PSL switch, and moves to the right or left. Moving the switch to the center position will keep the directional lights from blinking.

• MANUALLY TRANSPORTING YOUR AMIGO

Your Amigo can be disassembled and transported in most car trunks (Figure 9).

- 1. Turn OFF your Amigo.
- 2. Unlock the seat and remove it completely.
- 3. Remove the batteries.
- 4. Fold handle down to

- platform (See page 7).
- 5. Lift the rear into the trunk of the vehicle and slide the Amigo into the trunk.

Note: When batteries are removed, the wheels will not rotate for safety purposes.

- 6. Place seat and batteries into the trunk.
- 7. If your Amigo has a power seat lift, we suggest placing a cut out tennis ball or something similar over the seat post to protect it. If you leave the batteries connected, make sure the lift cannot be accidentally activated. The "freewheeling" switch must be in the "normal" position.

• LOAD YOUR AMIGO INTO YOUR CAR WITH A LIFT•ALL®

Note: For greater convenience, it is recommended you obtain a Lift*Alt* to assist you when loading and unloading. For more information on the Lift*Alt*, contact your local Amigo Mobility Center* or call toll-free: 1-800-248-9131. Use a van lift system only if it has a proper hold-down for the Amigo.

• AIRPLANE TRAVEL

An Airline Travel Kit (Item #300116) is available from Amigo Mobility International. It contains a battery holding strap, three "Special Handling" Tags and a "Helpful Hints for Airline Travel" Booklet. Non-spillable batteries are approved for travel on airplanes. When you make your reservation, notify the airline that you will be requesting an "Escort" service. This will assure you that your Amigo will be picked up and delivered at the boarding gate.

GROUND TRAVEL

Some public transportation systems have accommodations for your Amigo, but you must transfer to a seat.

AMIGO

by Marjorie Spencer Darling

I have a small electric cart
I always have in tow,
for every time I take a trip
Amigo's sure to go.
It followed me abroad this year,
it simply is terrific!
We went to the antipodes,
and to the South Pacific.
You, too, can take Amigo now
and travel many places.
No need for you
to stay at home - Go!
See new things and new faces.



Figure 9



Helpful Hints for Everyday Use



In the bathroom, maneuvering room is usually limited. Position close to the toilet, swivel the seat, lock it in position and transfer.

When using the bathtub, back up parallel to the tub. Swivel the seat and place your feet in the tub. Engage seat lock and carefully transfer.

∆WARNING

Do not use your Amigo while taking a shower. The water may get into the controller and cause electronic malfunctions.

• BEDROOM

To transfer to the bed, back up parallel to the bed. Swivel, lock seat into position, and ease

onto the bed. If charging the batteries overnight, it may be easiest to plug in the battery charger before transferring. For added ease, when using your power seat lift, raise your seat slightly higher than the surface from which you are transferring.

• KITCHEN

For access to a refrigerator, make a parallel approach or back up to the outside of the refrigerator. Swivel the seat.

Approach a table, sink, or stove by maneuvering parallel to it. Swivel the seat to the desired position. The handle will be out of the way.





• CAR

To transfer to a car, make a parallel approach to the side of the car, release the door latch, and open the door as you drive forward. Back parallel to the side of the car and into the door opening. Swivel the seat and lock it in position and carefully slide onto the car seat. On the driver's side you can use the steering wheel for leverage.

• RAMPS AND INCLINES

- Always lean forward as you maneuver your Amigo up a ramp or incline of an angle equal to 1" of rise to 12" of travel (length).
- Engage your seat lock when travelling up ramps or inclines.

• USING AN ELEVATOR

Backing into an elevator allows you to exit forward. Be sure the elevator is level with the floor.

• OPENING DOORS

When a door opens toward you, approach at an angle and just off to one side. Grasp the door knob, reverse and pull the door open. For doors that open away from you, position the bumper against the door, close to the doorjamb on the doorknob side of the door. After you release the latch, use the bumper to push the door open.

▲ CAUTION

BE CAREFUL WITH GLASS DOORS TO AVOID PERSONAL INJURY!

• KEEP YOUR AMIGO SHINING

Clean a fabric seat with upholstery cleaner or wash with mild soap and water. Wipe the naugahyde seat clean with a damp cloth. Protect the housings and all painted parts with a coat of auto wax.



AWARNING

Do not expose your Amigo to spray-on cleaning solutions and excessive moisture such as spraying it with a garden hose, shower, or leaving it uncovered in heavy rain.

Excessive moisture around the controller or electrical connections may cause contamination of electrical circuitry, damage and malfunction as well as personal injury.

• SPEED CONTROLS

Slower speeds are recommended indoors and while learning to operate your Amigo. Higher speeds can be used when conditions allow.

• STOPPING

- 1. Dynamic braking will gradually bring your Amigo to a stop on a level surface.
- 2. The Electromagnetic Brake will stop your unit one to two seconds after releasing thumb control.
- 3. IN EMERGENCY ONLY quick stopping with your Amigo can be accomplished by reversing the handle control in order to reverse the direction the Amigo is travelling.

▲ CAUTION

THIS PROCEDURE IS NOT RECOMMENDED.




■ MAINTENANCE

- Daily
- · Bi-Weekly
- Monthly
- 3 Months
- 6 Months
- Yearly
- Controller
- UNIPAC™ Drive

SERVICE

- Replacing Rear Drive Wheel Tire or Inner Tube
- **WARRANTIES**



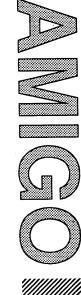





Maintenance



Minimum maintenance is required to increase your service ease and dependability. Check the following:

• DAILY

- Maintain battery charge.

• IF YOU USE YOUR AMIGO DAILY

 It is important to charge the battery on a daily basis. Charge the battery till fully charged to ensure battery operation throughout the cold weather months.

Storage Tip: Extended charging is good for the batteries. The batteries will not over-

charge. Charge battery fully and store indoors in an arid environment. Do not store on concrete since this will continue to draw on the battery. Rather, protect your battery by placing it on a wood block instead.

• MONTHLY

ALL RWDs:

- Check and clean battery terminals and the copper connections as needed with a light piece of emery cloth. Lightly pull copper spring tabs out 1/8" to 1/4" to insure successful connection when battery is replaced.
- Rotate batteries from one box to another to be sure to maintain a full charge in each battery.
- Clean upholstery, plastic and

- exposed fasteners.
- Check front wheel bearings for roughness during rotation. Replace if necessary.
- Make sure all fasteners are tight at all connections on battery box.

AMIGO DELUXE:

- Remove wide-track dual wheels and inspect interconnecting parts of the dual wheels and drive wheels for signs of dirt build up. Wipe clean and lightly coat interconnecting parts with lubricant or grease.
- Check tire pressure. All pneumatic tires on rear wheels should be maintained at 40-50 psi.

AWARNING

DO NOT EXCEED 50 PSI OR TIRES MAY BE DAMAGED!

• 3 MONTHS

ALL RWDs:

- Lubricate steering column and seat post.
- Check all bolts and clips holding wheels for tightness and wear.

•6 MONTHS - ALL RWDs

- Check all electrical connections for clean contact.
- Charging connection should be inspected for tight connection.
- Check tires for wear.
- Inspect seat posts for cracks or breaks in welds.

• YEARLY

 Have your authorized Service Center check your Amigo completely.

• CONTROLLER

Do not attempt to service the Controller or related electronic components as you may void your warranty.

• UNIPACTM DRIVE

Your UNIPAC™ Drive System is a precision assembly lubricated for long service. Do not attempt to open and service the assembly, which could void your warranty. Contact your authorized Service Center if problems occur or call 1-800-248-9131 to speak to a manufacturer service representative.







Service

You will get years of dependable service if you follow the suggested maintenance schedule. If you experience a problem and have to write in or call an authorized Service Center, be sure to provide your serial number. You will find it engraved on the front side of the first crossbar that is located underneath the platform of your Amigo and on the nameplate as well. Model numbers and serial numbers will help us in providing efficient service. Call your local Mobility Center, Amigo Representative, or call: 1-800-248-9131.

AWARNING

DO NOT ATTEMPT
TO DISASSEMBLE THE RIM
BEFORE RELEASING ALL AIR
PRESSURE FROM THE TIRE.

• REPLACING REAR DRIVE WHEEL TIRE OR INNER TUBE

RWD DELUXE:

Tools: Two 1/2" sockets, small screw driver, two long shank screwdrivers, Phillips screwdriver.

Disassembly:

- 1. Remove seat from the unit.
- Remove batteries and battery case cover (Amigo Deluxe and Amigo Sport).
- 3. Remove rear cover.
- 4. Remove dual wheels from the same side of the unit in which the tire needs repairing, or remove center cap from rim if the dual wheels are not in place.
- Remove shoulder bolt, washers, and nut holding rear wheel adapter assembly to axle. Slide assembly off axle, tapping with mallet if necessary.
- 6. If the tires are pneumatic and not foam-filled, release any air pressure from the tire by depressing the needle on valve with the tip of a

- small screwdriver or similar instrument (Be careful not to damage the valve).
- 7. When all the air has been released, the rim can be disassembled and removed from the wheel assembly by removing the four bolts and nuts which hold it together. This requires two 1/2" sockets.
- 8. If the rim does not easily disassemble after removing bolts and nuts, use two long shank screwdrivers to gently pry the tire from the rim. Be careful not to damage the inner tube of the tire.

Reassembly:

- 1. Release all air from the inner tube if tires are pneumatic.
- 2. Insert inner tube inside tire with valve stem pointing out.
- 3. Insert one rim half inside tire making sure the valve stem of the tube fits in the cutout on the inner face of the rim.
- 4. Put the two halves together (Do not pinch inner tube).
- 5. Insert the four bolts through the rim and adapter making sure the bolt heads are on the opposite side of

the valve stem.

- 6. Place the locknuts on bolts and tighten the assembly together.
- 7. Inflate tire to 40-50 psi.

Note: Replace spacer and shim washers. Then slide wheel assembly over axle, using a light oil lubricant if necessary. Re-install shoulder bolt, washers, and nut.

Warranties

AMIGO MOBILITY INTERNATIONAL, INC.

Effective September 1, 1990

• LIMITED THREE-YEAR WARRANTY (U.S.)

Amigo Mobility International, Inc. (AMI), warranties products it manufactures to the original purchaser.

ALL AMIGO MANUFACTURED PRODUCTS





1. Structural components that are warranted against defects in materials and workmanship for three (3) years are limited to the following components:

Platform, fork, lower seat post and upper seat post.

- 2. UNIPAC™, vari-speed boards and motors are warranted against defects in materials and workmanship for three (3) years with replacement on a pro-rated basis.
- 3. The RWD Heavy Duty is warranted against defects in materials and workmanship according to 4 and 5 below. There is no warranty of a power seat lift if used on an RWD Heavy Duty.
- 4. Components warranted for six (6) months against defects in materials and workmanship are limited to the following:

LED lights, electrical and mechanical switches, and batteries provided through AMI.

5. Chargers, seats and other components have a one (1) year warranty and are covered against defects in materials and workmanship for one year from date of purchase.

- 6. Use of this product by persons exceeding 250 lbs. on all units (except RWD Heavy Duty which is 350 lbs.) and 200 lbs. on the Lift*All® is not recommended and voids this warranty.
- 7. After six (6) months, labor will be charged on warranty repairs.

AMI's obligation under this warranty is limited to replacing or repairing at its option, such products or parts that are returned prepaid by the purchaser and are found to be defective.

If the purchaser elects not to return the unit to the factory and wishes a factory service representative to make adjustments and repairs at the unit's location, field service rates will apply.

This warranty does not cover accidental damage, misuse, normal wear and tear, or use of Amigo products for other than personal purposes.

The three-year warranty pro-rating is as follows: (Customer pays)

- 01 12 months = 0
- 13 24 months = 50%
- 25 36 months = 75%

The above warranty supersedes and is in lieu of all other warranties, expressed or implied, and no person, agent or dealer is authorized to give any warranties on behalf of the manufacturer, nor to assume for the manufacturer any other liability in connection with any of its products unless made in writing and signed by an official of the manufacturer.

This warranty provides you with specific legal rights, with the possibility of other rights, which vary depending upon your locality.

A one (1) year warranty is applicable outside the United States, with the exception of batteries, LED lights, and switches, which have a six (6) month warranty.

• FOR WARRANTY SERVICE

Contact: Service Department of AMI, 6693 Dixie Highway, Bridgeport, MI 48722, TOLL-FREE: **1-800-248-9131**. For emergencies, contact your local Amigo Mobility Center[®], Amigo Representative, or after business hours, use the 24-Hour Service Hotline: **1-800-248-9131**.

if you have any further questions regarding your Amigo, need further assistance in troubleshooting a problem, or if your Amigo needs service or repair, please call the Amigo Service Department at 1-800-248-9131. A complete Service Guide and Electrical Troubleshooting Kitare available to you. For further information, call our hotline toll-free: 1-800-248-9131.



Available Accessories for the Amigo Centra, Amigo Deluxe, Amigo Sport and Amigo IV!

The accessories shown here are available from Amigo Mobility International, Inc. The order can be placed through your local Amigo Representative, Amigo Mobility Center[®], or by calling 1-800-248-9131. To help you save money when purchasing theses items, please consider using the "Amigo Gold" referral program.

■ Basket and Tote Bag

Carry those "necessities" in an Amigo wire mesh basket, which conveniently attaches to the handle of your Amigo. Also available is a durable canvas tote bag which fits inside the basket. It keeps small articles from slipping through the mesh.

■ Battery

Have all those miles taken a toll on your battery? Order a new battery to keep on hand ... just in case! Comes with a full six-month warranty. Note: Charge your battery once a month when it is being stored.


■ Power Seat Lift

With this lift on your Amigo, shelves and counters are easier to reach. Powered by the Amigo battery, the touch of a switch gives you a lift.


■ Sheepskin Cover

Add a touch of luxurious comfort to your Amigo with all natural sheepskin. It not only looks good, but it's soft, adds cushioning, and is extremely durable.

See the referral card page in the back of this manual for further information on how to "Go for the Gold" and save money!

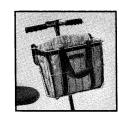



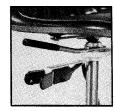





SERVICE KITS

Listed here are just a few of the Service Kits that are available for your Amigo.

#100672 - Seat Post Repair Kit

A Seat Post repair kit to keep your seat in top shape.

Includes:

- (1) Shoulder Bushing
- Collar Stem
- 1) Lower Seat Post Bearing
- 1) 5/16 18 Hex Keps Nut 1) .125 x 1.125 Roll Pin
- (1) 5/16 18 x 2.50 Hex Head Bolt



■ RWD Travel Kits

These include everything you will need to travel with your Amigo IV, Sport, RWD, Deluxe, and Centra.

#300309 - RWD w/Belt Drive Kit Includes:

- (3) Special Handling Tags
- (1) Airline Strap
- (1) Belt
- (1) Tube Repair Kit
- (1) Service Guide

#300310 - RWD w/UNIPAC Kit Includes:

- (3) Special Handling Tags
- Airline Travel Strap
- 1) Tube Repair Kit
- (1) Service Guide



■ #300367 Electrical Diagnostic Kit

This kit contains all the necessary items that will allow you to determine the source of any electrical problems that your Amigo may incur.

Includes:

- (5) Test Pluas
- (1) Electrical Troubleshooting Guide





This book provides a with quick reference that allows you to begin to find the source of any service problems you may encounter.





The Service Kits shown here are available from Amigo Mobility International, Inc. The order can be placed through your local Amigo Representative, Amigo Mobility Center®, or by calling 1-800-248-9131. To help you save money when purchasing theses items, please consider using the "Amigo Gold" referral program.

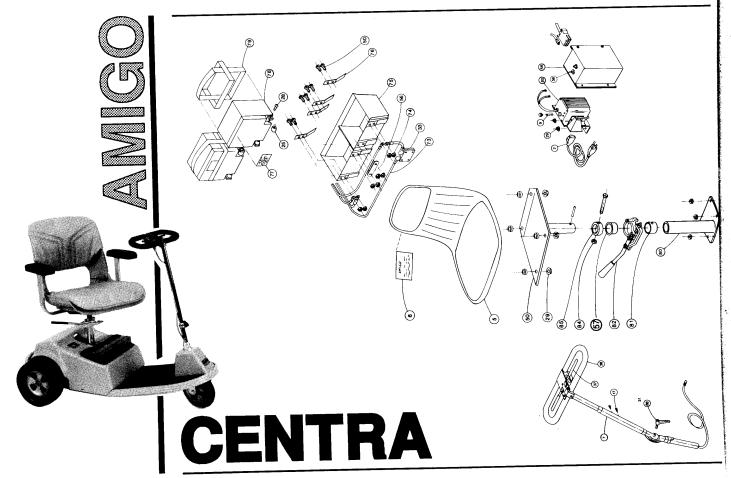

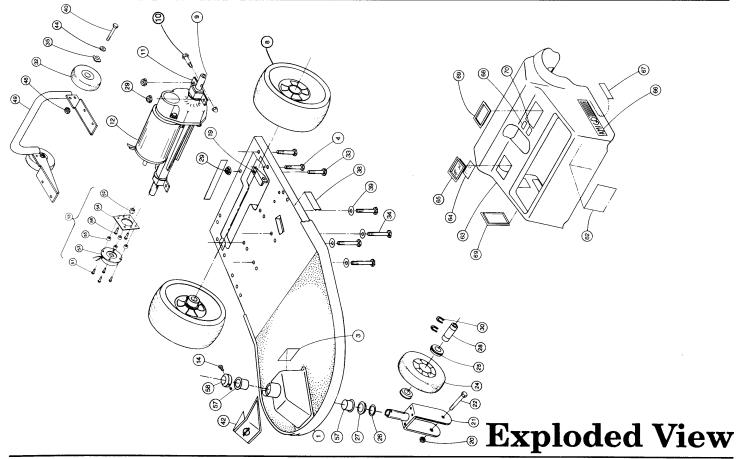


Amigo Centra Parts List


Ref#	ltem#	Description
1.	300481	Platform Assembly - Fiberglass RWD
		- (Includes #75)
2.	7150	Reel - Enclosed w/o Ball
		- (Includes #119)
	1281	Label - Owner's
4.	1621	Boit - 1/4-20 x 2.0 Hex HD
5.	100660	= · · · · · · · · · · · · · · · ·
		- (Includes #6)
6.	1295	Label - Seat Back
8.	300237	
9.	6993	Bracket - Charger Reel
10.	5865	Screw - 10 - 24 x 1/4 x 1.25 Shoulder
	5751	
12.	300409	UNIPAC™ w/Brake - Sport
		- (Includes #13)
13.	300437	
		- (Includes #s 51 - 55, 61)
14.	6230	Screw, Retaining Collar
15.	300462	
		- (Includes #s 16, 35, 40, 44, 45, 48)
16.	300461	Tube - Fiberglass Base Bumper
17.	5889	Screw - 8/32 x .625 Button HD
18.	5206	Pin - 3/16 x 2 Groove
19.	5753	Strap - Torque
20.	2237	Nut - 3/8 - 24 Hex
21.	5172	Fork - RWD (8/18/87)
22.	2236	Bolt - 3/8 - 24 x 3.75 Hex HD
23.	1024	Spacer - Steering Column

24. 300034	•
OF 10FF	- (Includes #'s 25, 28, 30)
25. 1055 26. 5413	Bearing - Wheel Washer - Dust Cover
27. 5407	
	Bearing - Steering Column Ball
28. 5701	Centering Tube - RWD Nut - 1/4 - 20 Serrated
29. 3083 30. 1040	
30. 1040	Retaining Ring - Centering Tube
31. 6461	Lever - Adjustable Handle
32. 6146	Contact - Battery Jumper
33. 1344	Screw - 1/4 - 20 x 1.50 Hex HD
34. 4954	Screw - 1/4 - 20 x 2.0 Button HD
35. 5340	Wave Washer
36. 300288	Grip - Adjustable Handle Vinyl
37. 5719	Cover - Pair of Adjustable Handle Tab
38. 6304	Velcro - Rear Cover/Base
39. 5893	Washer25 I.D. Fender
40. 1824	Bolt - 1/4 - 20 x 2.5 Hex HD
42. 6926	Label - RWD Light Blue Front Logo
6927	Label - RWD Navy Blue Front Logo
6928	Label - RWD Maroon Front Logo
6929	Label - RWD Green Front Logo
43. 1391	Screw - 10 - 20 x .50 Button HD
44. 1192	Washer25 I.D. Flat
45. 300008	
	- (Includes #'s 35, 40, 44, 48)
46. 6193	Circuit Breaker
47. 5200	Terminal Contact
48. 3622	Nut - 1/4 - 20 x .50
49. 7145	Seal, Rubber - E/M Brake
51. 6899	Screw - #10 - 32 x 5/8 Socket HD Cap
52. 5687	Hub - RWD E/M Brake Adapter
53. 300243	Brake - Electromagnetic

55 56 57 58 59 60 61 62 63	300348 1025 300304 5825 7149 111576 5806 300466 300467 300468 300469	Bearing - Shoulder Collar Kit - Retaining Switch - E/M Brake Release Charger - Sport BF#8762DN/RWD Screw - 10 - 32 x .5 Self-Tapping Label - RWD Warning Cover - Rear Green Cover - Rear Maroon Cover - Rear Rich Blue Cover - Rear Navy Blue
	5807	Label - Charger
	200021	Bezel - Display (Window)
66.		Label - Left Side Amigo
	6749	Label - Right Side Amigo
	6305	Velcro - Rear Cover/Base
	5944	Label - E/M Brake Release
	7122	Grommet - Reel Cord
	5827	Label - E/M Brake
	3973	Screw for Battery Contact
	300165	Wire Assembly - Battery (11/87)
	1167	Nut - 1/4 - 20 Hex Keps
75.	300121	Holder Assembly - Dual Battery - (Includes #32)
76.	5204	Contact - Dual Power Cell
77.	1474	Label - Battery
78.	300014	Battery - RWD
79.	300015	- (Includes #77) Harness - Battery Lifting

80. 300265 Lower Seat Post 81. 100710 Bearing - Lower w/Pin 82. 267000 Seat Lock Assembly - RH 267500 Seat Lock Assembly - LH 84. 267001 Stem Collar Bolt & Nut 85. 100690 Stem Collar with Bolt & Nut - (Includes #84) 86. 100670 Upper Seat Post 88. 300479 Handle - Centra Telescopic - (Includes #s 31, 17) 119. 7123 Screw - 10 - 24 x .50 120. 4750 Screw - 10 - 32 x 5/8 TRS HD 121. 7201 Bushing

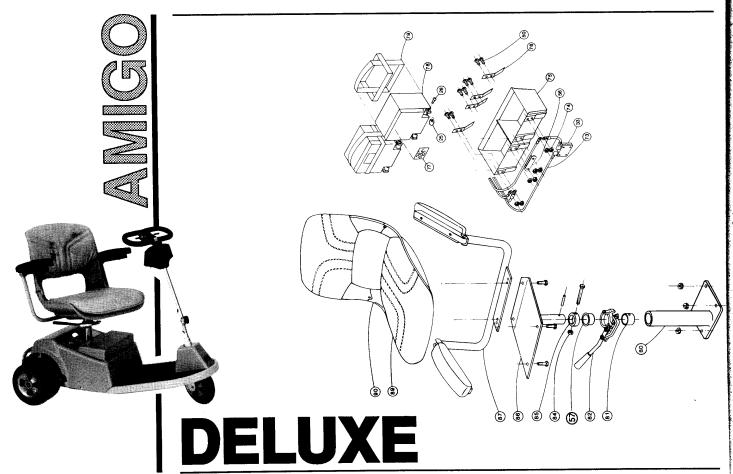


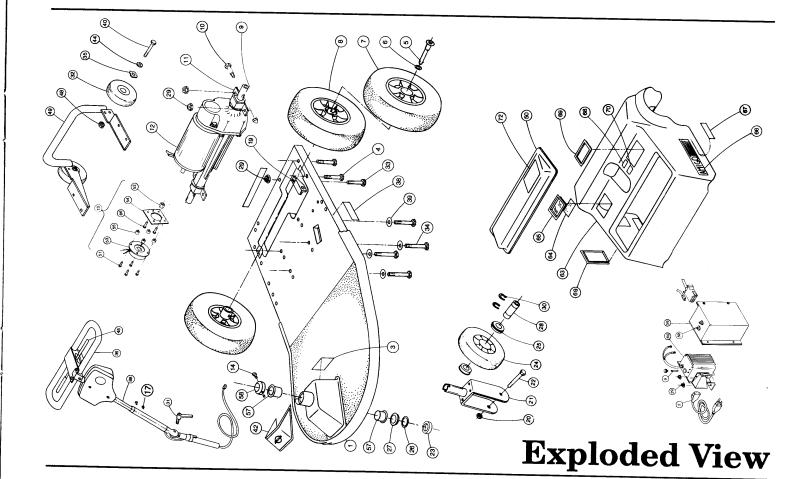


Amigo Deluxe Parts List

An	nigo	Deluxe Parts List	24.	300034	Wheel Assembly - RWD Front - (Includes #s 25, 28, 30)
			25.	1055	Bearing - Wheel
			26.	5413	Washer - Dust Cover
Ref#	ltem#	Description	27.	5407	Bearing - Steering Column Ball
1.	300481	Platform Assembly - Fiberglass RWD	28.	5701	Centering Tube - RWD
		- (Includes #75)	29.	3083	Nut 1/4 - 20 Serrated
2.	7150	Reel - Enclosed w/o Ball	30.	1040	Retaining Ring - Centering Tube
		- (Includes #119)	31.	6461	Lever - Adjustable Handle
3.	1281	Label - Owner's	32.	6146	Contact - Battery Jumper
4.	1621	Bolt - 1/4 - 20 x 2.0 Hex HD	33.	1344	Screw - 1/4 - 20 x 1.50 Hex HD
5.	5432	Pin - 1.5 in Dual Wheel	34.	4954	Screw - 1/4 - 20 x 3.0 Button HD
6.	5434	Washer - RWD Pin Shim	35.	5340	Wave Washer
7.	300370	Wheel - RWD Plastic Dual	36.	300288	Grip - Adjustable Handle Vinyl
8.	300371	Wheel - RWD Plastic Rear	37.	5719	Cover - Pair of Adjustable Handle Tab
9.	6993	Bracket, Charger Reel	38.	6304	Velcro - Rear Cover/Base
10.	5865	Screw - 10 - 24 x 1/4 x 1.25 Shoulder	39.	5893	Washer25 I.D. Fender
11.	5751	Clamp - Mounting	40.	1824	Bolt - 1/4 - 20 x 2.5 Hex HD
12.	300409	Drive System - UNIPAC™	42.	6926	Label - RWD Light Blue Front Logo
		- (Includes #13)		6927	Label - RWD Navy Blue Front Logo
13.	300355	Brake Assembly - RWD E/M		6928	Label - RWD Maroon Front Logo
		- (Includes #s 51 - 55, 86)		6929	Label - RWD Green Front Logo
14.	6230	Screw, Retaining Collar	43.	1391	Screw - 1/4 - 20 x .50 Button HD
15.	300462		44.	1192	Washer25 I.D. Flat
		- (Íncludes #s 16, 35, 40, 44, 45, 48)	45.	300008	Wheel - Bumper
16.	300461				- (Includes #s 35, 40, 44, 48)
17.	5889	Screw - 8 - 32 x .625 Button HD	46.	6193	Circuit Breaker
18.	5206	PIN 3/16 x 2 Groove	47.	5200	Terminal Contact
19.	5753	Strap - Torque	48.	3622	Nut - 1/4 - 20 Hex Ny - Lok
	2237	Nut - 3/8 - 24 Hex	49.	7145	Seal, Rubber - E/M Brake
	5172	Fork - RWD (8/18/87)	50.		Cover Assembly
	2236	Bolt - 3/8 - 24 x 3.75 Hex HD		300209	- Battery Top Gray
	1024	Spacer - Steering Column		300213	- Battery Top Green
					<u> </u>

Battery Top Rich Blue							
Screw - #10 - 32 x 5/8 73 300165 Socket HD Cap 74 1167 Socket HD Cap 74 1167 Socket HD Cap 75 300121 Socket HD Cap 74 1167 Socket HD Cap 75 300121 Socket HD Cap Socket HD			300215 300216	- Battery Top Maroon - Battery Top Navy Blue	70	. 5827	Label - E/M Brake
Socket HD Cap		51.	6899	Screw - #10 - 32 x 5/8			
52. 5687	i						
53. 300243 Brake - Electromagnetic 54. 5811 Plate - RWD E/M Brake Adapter 55. 5832 Spacer - RWD E/M Brake 56. 300348 Controller Assembly	- {			Hub - RWD E/M Brake Adapter			THE TO TICK REDS
Same		53.	300243	Brake - Electromagnetic	13.	300121	
Spacer - RWD E/M Brake				Plate - RWD E/M Brake Adapter	76	E204	- (Includes # 32)
Sect Controller Assembly	i	55.	5832	Spacer - RWD E/M Brake			Contact - Dual Power Cell
- Amigo Sport	1	56.	300348	Controller Assembly			
57. 1025 58. 300304 Collar Kit - Retaining Jet and Shopper 59. 5825 Switch - E/M Brake Release 60. 7149 Charger - Sport BF#8762DN/RWD 61. 6223 Screw - 10 - 32 x .5 Self Tapping 63. 300466 Cover - Rear Green - Rear Maroon 300468 300469 - Rear Rich Blue 300469 - Rear Gray - (Includes #s 49, 50, 62 - 70) 64. 5807 65. 200021 66. 5326 - Label - Charger - Sport Bear Rich Side Amigo 67. 6305 Velcro - Rear Cover/Base 68. 5944 - Label - E/M Brake Release				- Amigo Sport	70. 70	300014	Baπery RWD - (Includes #77)
Sect Component Collar Kit Retaining Jet and Shopper Switch - E/M Brake Release Switch - E/M Verw - Switch - E/M Verw - Switch - E/M Verw - Elin Component - 24 Volt				Bearing - Shoulder	79.	300015	Harness - Battery Lifting
Second Second Shopper Second Shopper Second Shopper Switch - E/M Brake Release Second Seco		58.	300304	Collar Kit	60.	300137	Power Seat Lift Component
Second Section				- Retaining Jet and Shopper	01	100710	- 24 Volt
60. 7149 Charger		59.	5825	Switch - E/M Brake Release	01. 02	267000	Bearing - Lower w/Pin
- Sport BF#8762DN/RWD 61. 6223		60.	7149	Charger	02.	267000	Seat Lock Assembly - RH
61. 6223	-				0.4	207004	Seat Lock Assembly - LH
62. 5806		61.	6223	Screw - 10 - 32 x .5 Self Tapping	04. 0F	100000	Stem Collar Bolt & Nut
63. 300466 Cover		62.	5806	Label - RWD Warning	65.	100690	Stem Collar with Bolt & Nut
- Hear Green - Rear Maroon - Rear Maroon - Rear Rich Blue - Rear Navy Blue - Rear Gray - (Includes #s 49, 50, 62 - 70) - Clock Save Label - Charger - Clock Amigo - Cushion Sets - High Back Gray Folding - Cushion Sets - High Back Gray Folding - Cushion Sets - High Back Gray Folding - High Back Black Folding - Cushion Sets - High Back Black Folding - High Back Black Folding - Cushion Sets - High Back Black Folding - Cushion Sets - High Back Black Folding - Cushion Sets - High Back Black Folding - High Back Black Folding - Cushion Sets - High Back Black Folding - High Back Black Folding - Cushion Sets - High Back Black Folding - High Back Black Folding - Cushion Sets - High Back Black Folding - How Back Black Black Black Folding - How Back Black Bl		63.	300466	Cover	96	100074	- (Includes #84)
300468 - Rear Rich Blue 300469 - Rear Navy Blue 300470 - Rear Gray - (Includes #s 49, 50, 62 - 70) 64. 5807 Label - Charger 65. 200021 Bezel - Display (Window) 66. 5326 Label - Left Side Amigo 67. 6305 Velcro - Rear Cover/Base 68. 5944 Label - E/M Brake Release 67. 6805 Care Rear Cover/Base 68. 5944 Care Rich Blue 87. 300183 (Optional) Arms - Flip-Up - Folding Seat Handle - Deluxe Telescopic - (Includes #s 31, 36, 37) Seat - High Back Gray Folding Seat - High Back Gray Folding Cushion Sets - High Back Gray Folding - High Back Black Folding - High Back Black Folding - Screw - 10 - 24 x .50 - Screw - 10 - 32 x 5/8 TRS HD					00.	1006/1	
300468 - Rear Rich Blue 300469 - Rear Navy Blue 300470 - Rear Gray - (Includes #s 49, 50, 62 - 70) 64. 5807 Label - Charger 65. 200021 Bezel - Display (Window) 66. 5326 Label - Left Side Amigo 67. 6305 Velcro - Rear Cover/Base 68. 5944 Label - E/M Brake Release 67. 500183 (Optional) Arms - Flip-Up - Folding Seat Handle - Deluxe Telescopic - (Includes #s 31, 36, 37) Seat - High Back Gray Folding 300369 Seat - High Back Black Folding Cushion Sets - High Back Gray Folding - High Back Black Folding - High Back Black Folding - Screw - 10 - 24 x .50 - Screw - 10 - 32 x 5/8 TRS HD	1		300467	- Rear Maroon	07	200100	- Power Seat Lift Upper
300469 - Rear Navy Blue 300470 - Rear Gray - (Includes #s 49, 50, 62 - 70) 64. 5807	ı		300468	- Rear Rich Blue	07.	300183	
300470 - Rear Gray - (Includes #s 49, 50, 62 - 70) 64. 5807	ł		300469	- Rear Navy Blue	00	200470	- Folding Seat
- (Includes #s 49, 50, 62 - 70) 64. 5807	ĺ		300470	- Rear Gray	00.	300478	
64. 5807 Label - Charger 65. 200021 Bezel - Display (Window) 66. 5326 Label - Left Side Amigo 6749 Label - Right Side Amigo 67. 6305 Velcro - Rear Cover/Base 68. 5944 Label - E/M Brake Release 69. Seat - High Back Black Folding 70. Cushion Sets 70. 300361 - High Back Gray Folding 70. 300361 - High Back Black Folding 7123 Screw - 10 - 24 x .50 7120. 4750 Screw - 10 - 32 x 5/8 TRS HD				- (Includes #s 49, 50, 62 - 70)	90	200210	- (Includes #s 31, 36, 37)
65. 200021 Bezel - Display (Window) 66. 5326 Label - Left Side Amigo 67. 6305 Velcro - Rear Cover/Base 68. 5944 Label - E/M Brake Release 90. Cushion Sets 300220 - High Back Black Folding 300361 - High Back Black Folding 300361 - High Back Black Folding 119. 7123 Screw - 10 - 24 x .50 120. 4750 Screw - 10 - 32 x 5/8 TRS HD	1			Label - Charger	09.		
66. 5326 Label - Left Side Amigo 300220 - High Back Gray Folding 300361 - High Back Black Folding 300361 - High Back Black Folding Screw - 10 - 24 x .50		65.	200021	Bezel - Display (Window)	00	300369	
6749 Label - Right Side Amigo 300361 - High Back Gray Folding 300361 - High Back Black Folding 119. 7123 Screw - 10 - 24 x .50 Label - E/M Brake Release 120. 4750 Screw - 10 - 32 x 5/8 TRS HD		66.	5326	Label - Left Side Amigo	90.	200220	
67. 6305 Velcro - Rear Cover/Base 119. 7123 Screw - 10 - 24 x .50 120. 4750 Screw - 10 - 32 x 5/8 TRS HD	ĺ		6749	Label - Right Side Amigo			- High Back Gray Folding
68. 5944 Label - E/M Brake Release 120. 4750 Screw - 10 - 24 x .50		67.	6305	Velcro - Rear Cover/Base	110		- High Back Black Folding
120. 4750 Screw - 10 - 32 x 5/8 TRS HD		68.					Screw - 10 - 24 x .50
Busning							Screw - 10 - 32 x 5/8 TRS HD
					121.	7201	busning

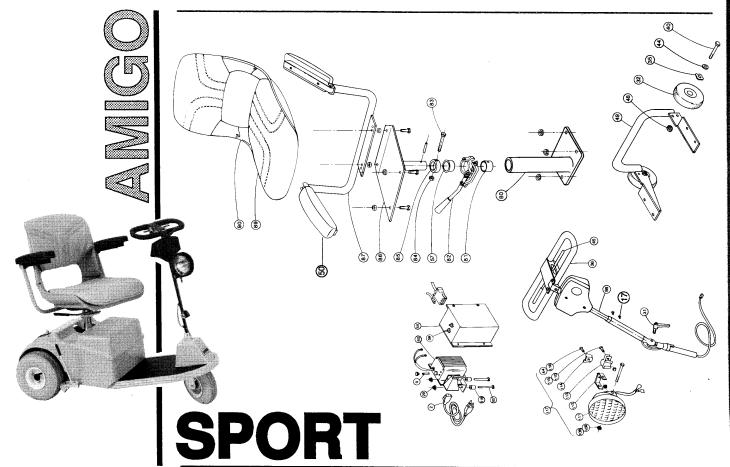

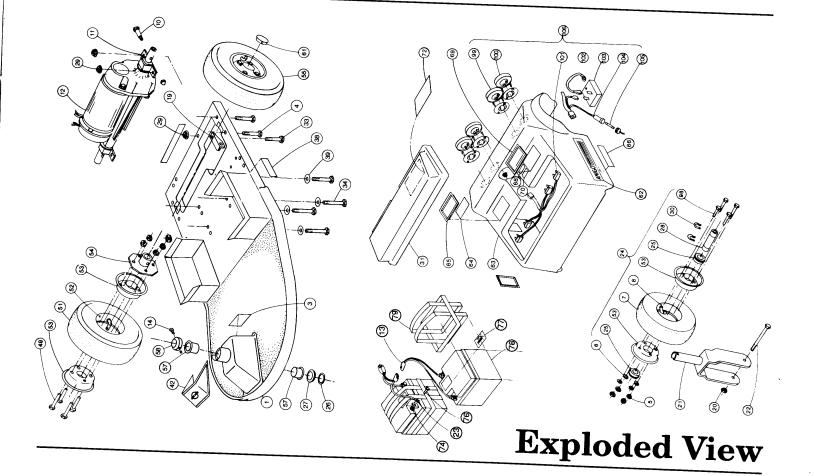

Amigo Sport Parts List

Ref# Item#	
2. /150	
- (Includes #119) 3. 1281 Label - Owner's 4. 1621 Bolt - 1/4 - 20 x 2.0 Hex HD	
3. 1281 Label - Owner's 4. 1621 Bolt - 1/4 - 20 x 2.0 Hex HD	
4. 1621 Bolt - 1/4 - 20 x 2.0 Hex HD	
5 1275 Nut - 5/16 - 10 Hay	
6. 1259 Washer - 5/16 I.D. Spring Lock	
/. 5181 Tire - Gray - 2.80 - 2.50 - 4 Pneumatic	
8. 2725 Tube - Pneumatic Tire	
9. 6993 Bracket - Charger Reel	
10. 5865 Screw - Shoulder - 10 - 24 x 1/4 x 1.25	
11. 5751 Clamp - Mounting	
12. 300409 Drive System - UNIPAC™ - (Includes #19)
13. 300355 Brake Assembly - RWD (E/M)	,
- (Includes #s 51 - 55, 86)	
14. 6230 Screw - 1/4 - 28 x .50 - LG SLT Fillister H	1D
15. 300462 Bumper Assembly - Fiberglass Base	
- (Includes #s 16, 35, 40, 44, 45, 74)	
16. 300461 Tube - Fiberglass Base Bumper	
17. 5889 Screw - 8-32 x .625 Button HD	
18. 5206 Pin - 3/16 x 2 Groove	
19. 5753 Strap - Torque	
20. 2237 Nut - 3/8 - 24 Hex	
21. 6433 Fork - Amigo Sport RWD	
22. 6505 Bolt - 3/8 - 24 x 4 5 Hex HD	
23. 300440 Wire - Sport Battery #BF09735UP	
24. 300352 Wheel Assy 8" Pneumatic - RWD Fron	t
- (Includes #s 5-8, 25, 28, 30, 43, 98)	•
25. 1055 Bearing - Wheel	
26. 5413 Washer - Dust Cover	-
27. 5407 Bearing - Steering Column Ball	
J =	

28. 6433 29. 3083 30. 1040 31. 6461 32. 300441 33. 2822 34. 4952 35. 5340 36. 300288 37. 5719 38. 6304 39. 5893 40. 1824 41. 5209 42. 6926 43. 1258 44. 1192 45. 300008	Centering Tube - Amigo Sport Nut 1/4 - 20 Serrated Retaining Ring - Centering Tube Lever, Adjustable Handle Wire - Sport Battery #BF09735UP Bolt, 1/4 - 20 x 1.75 Hex HD Screw - 1/4 - 20 x 2.00 Button HD Wave Washer Grip - Adjustable Handle Vinyl Cover - Pair of Adjustable Handle Tab Velcro - Base Washer25 I.D. Fender Bolt - 1/4 - 20 x 2.5 Hex HD Cap - Rear Wheel Cover Label - RWD Rich Blue Front Logo Bolt - 5/16 - 18 x .75 Hex HD Washer25 I.D. Flat Wheel - Bumper
48. 3622 49. 7145 50. 300377 51. 6899 52. 5687 53. 300243 54. 5811 55. 5832 56. 300348 57. 1025 58. 300304 59. 5825 60. 7149 61. 6223 62. 5806 63. 300380 300381	- (Includes #s 35, 40, 44, 48, 74) Nut - 1/4 - 20 Hex Ny - Lok Seal - Rubber, E/M Brake Cover Assembly - Battery Top Sport Screw - #10 - 32 x 5/8 Socket HD Cap Hub - RWD E/M Brake Adapter Brake - Electromagnetic Plate - RWD E/M Brake Adapter Spacer - RWD E/M Brake Controller Assembly - Amigo Sport Bearing - Shoulder Collar Kit - Retaining Switch - E/M Brake Release Charger - Amigo Sport XTENDER™ - (Includes #'s 2, 9) Screw - 10 - 32 x .5 Self-Tapping Label - RWD Warning Cover - Sport Rear Cover - Sport Rear

	5807	Label - Charger	97.	6166	Tube - 10" Pneumatic Tire
	200021		98.	4615	Hub- Split Rim
66.	5326	Label - Left Side Amigo		6429	Adapter - Split Rim Rear Wheel
	6749	Label - Right Side Amigo		6455	Light - Turn Signal Yellow
67.	6305	Velcro - Rear Cover		300392	Wire Poor Cover Limbte
	5944	Label - E/M Brake Release			
	7122			6473	Wire - Battery to Flasher
	5827	Grommet, Reel Cord		6466	
		Label - E/M Brake	104.	300395	Wire - Battery to Fuse
72.	6464	Light - Turn Signal Red		6488	Fuse - 32 Volt 5 amp
/3.	1264 1167	Nut - 10 - 32 - Hex Keps	106.	300397	Cover Assembly - Sport Rear
<i>/</i> 4.	1167	Nut - 1/4 - 20 Hex Keps			- (Includes #s 49, 50, 62 - 70)
75.	300351	Wheel Assembly - 10" Pneumatic Rear		300398	Cover Assembly - Sport Rear w/Lights
		- (Includes #s 94 - 99)			- (Includes #s 49, 50, 62 - 70, 72, 100 - 105)
76.	1178		108	1253	Nut - 5/16 - 18 Hex Keps
77	1474	I abol - Ratton	400	1000	D-6 C40 40 0 CH HD
78.	300434	Battery - Sport Serial #BE09735UP	110	2272	Sorow 10 22 v 50 DD UD
		Battery - Sport Serial #BF09735UP - (Includes #77) Harness - Sport Battery #BF09735UP Seat Post - Lower - (Includes #s 29, 34) Bearing - Lower w/Pin Seat Lock Assembly - RH Seat Lock Assembly - LH Bolt - 5/16 - 18 x 2.50 Hex HD Stem Collar Bolt & Nut - (Includes #83)	110.	200200	Screw - 10 - 32 x .50 RD HD
70	6718	Harnes - Sport Botton, #DE0073511D	111.	300399	Headlight - Sport
90	200265	Soot Poot Lower (Includes #6.00.04)	112.	6479	Bracket - Sport Headlight
00.	100203	Dearing Lower - (Includes #\$ 29, 34)	113.	6484	Clip - Sport Headlight Handle
01.	00710	Dearing - Lower W/Pin	114.	5590	Clip - Sport Headlight Bracket
82.	26/000	Seat Lock Assembly - HH	115.	1480	Nut - 10 - 32 Hex Acorn
	26/500	Seat Lock Assembly - LH	116.	3168	Screw - 10 - 32 x .38 Phil Pan HD
83.	1450	Bolt - 5/16 - 18 x 2.50 Hex HD	117.	300349	Lighting Pkg w/unit Amigo Sport
84.	267001	Stem Collar Bolt & Nut - (Includes #83)			- (Includes #s 73, 108 - 113)
85.	100690	Stem Collar with Bolt & Nut - (Includes #84) Upper Seat Post - (Includes #'s 81, 85) Arms - Flip-Up - (Optional)	119.	7123	Screw - 10 - 24 x .50
86.	100670	Upper Seat Post - (Includes #'s 81, 85)	120	4750	Screw - 10 - 32 x 5/8 TRS HD
87.	300183	Arms - Flip-Up - (Optional)	121	7201	Bushing
88.	300478	Handle - Deluxe Telescopic	121.	7201	Bushing
	0000	(Includes #s 17, 31, 36, 37)			
QQ	300219	(Ontional) Seat High Book Crow Folding			
00.	300369				
	300309	(Optional) Seat - High Back Black Folding			·
00	000000	- (Includes #90)			
90.	300220				
	300361	(Optional) Cushion Set - High Back Black Folding			
	5567	Bolt - 5/16 - 18 x 2.0 Hex HD			
	3603	Nut - 5/16 - 18 Hex Ny-Lok			
96.	6167	Nut - 5/16 - 18 Hex Ny-Lok Tire - Gray 10" Rear Pneumatic			
		•			

